



SVANOVIR® FMDV 3ABC-Ab ruminant

Controlling the Foot and Mouth disease in ruminant populations

SUMMARY | The SVANOVIR® FMDV 3ABC-Ab ruminant test is based on non-structural 3ABC antigen giving you the advantages of early detection of antibodies, the discrimination between antibodies generated by field infection from those by vaccination as well as the identification of infected carriers in vaccinated populations.



YOUR CHALLENGE is a highly contagious virus with numerous serotypes

Foot-and-mouth disease virus (FMDV) causes an acute vesicular disease in cloven-hoofed animals resulting in impaired performance and loss of yield. There are seven different serotypes of the FMD virus which could vary within geographical regions. All of them are highly contagious and may lead to devastating outbreaks in naive populations. In some countries FMDV has become endemic.

YOUR GOAL is a fast and reliable identification of infected animals

The control of Foot-and-mouth disease (FMD) spread is achieved by the stamping out of infected animals, sanitation of contaminated premises and possibly vaccination of herds in affected regions. Fast and reliable identification of infected animals in susceptible and vaccinated populations are key aspects to avoid transmission and import of the disease.

ASSAY OVERVIEW

SVANOVIR® FMDV 3ABC-Ab ruminant

Species	Bovine, ovine and caprine		
Samples	Serum and plasma		
Type	Indirect ELISA based on non-structural 3ABC protein		
Article number	Plates	Tests	Samples
104895	10	480	440



YOUR SUPPORT

From 9am-4pm CET call:

 **+46 18 65 49 15**

 **customer.service@svanova.com**

Find more products and search for sales partners at:

www.svanova.com

Work effective protocol – results in less than 2 hours

Easy adaption to automatisation

User friendly

Manufactured under strict ISO 9001:2008 standardised procedures in Sweden



The SVANOVIR® FMDV 3ABC-Ab ruminant test provides accurate results in outbreak situations and for control- and eradication programs

Detects antibodies to all seven serotypes of the FMD virus

Discriminates between vaccinated and naturally infected animals

Detects carrier animals in vaccinated populations


High analytical sensitivity – early detection of infected animals

Reliable result – validated and used in several reference labs worldwide

PERFORMANCE CHARACTERISTICS SVANOVIR® FMDV 3ABC-Ab ruminant

A large number of tested individuals in different regions worldwide demonstrate the strong performance of the test. The high sensitivity provides for early detection – as early as 8 days post exposure. The excellent specificity minimises false positive test results which reduces the onset of unnecessary costly measures and trade embargos. The test discriminates reliably between antibodies from vaccinated and field infected animals. Moreover it identifies carriers in vaccinated populations.

	Sera from cattle South America, Europe and Middle East	Sera from small ruminants Europe
Specificity		
Disease free populations n=1424 n=134	97.0-99.8%	98.5%
Vaccinated populations n=619	98.5-100%	n.a.
Sensitivity		
Experimentally infected n=234	98-100%	n.a.

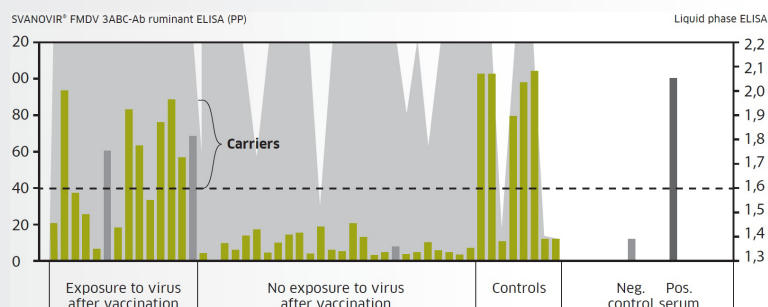

SVANOVIR® FMDV 3ABC-Ab ruminant ELISA identifies carrier animals in vaccinated populations. Figure shows antibody levels to structural proteins (▲) after vaccination as measured by liquid phase ELISA and to non-structural proteins (■) after exposure to field virus.

PSFMDV/02

Boehringer Ingelheim Svanova
Box 1545
SE-751 45 Uppsala, Sweden

www.svanova.com

svanova

COMPLEMENTARY DIAGNOSTIC PRODUCTS for Foot and Mouth Disease

Virus detection	Antibody detection	Quality control
SVANODIP® FMDV-Ag	SVANOVIR® FMDV 3ABC-Ab ruminant	IAEA FMDV standard sera